

# Maintenance and Adjustments



## Routine Maintenance

Suggested maintenance to be performed on unit. Contact your Authorized Penner Spa Distributor for periodic updates to this revision.

**Tub Model #** \_\_\_\_\_ **Tub Serial #** \_\_\_\_\_  
**Reservoir Model #** \_\_\_\_\_ **Reservoir Serial #** \_\_\_\_\_  
**Transfer Model#** \_\_\_\_\_ **Transfer Serial #** \_\_\_\_\_

Checklist	Yearly	
Check hoses and connections for leaks or damage.		
Check Aqua-Aire functions and controls.		
Check disinfectant system		
Check Thermometers accuracy with calibrated probe thermometer.		
Inspect electrical connections. ( GFCI, Circuit breakers)		
Check mixing valve for proper temperature. Use a calibrated probe thermometer to verify.		
Check Screens and water supply connections.		
Inspect Batteries for Transfers		
Check E-Stop to make sure it functions properly		
Check function of Emergency lowering button		
Belts for transfers and lifts should be free from tears.		
Castors should be kept clean and make sure the brakes hold securely.		
Hand control is functioning correctly		
Batteries for Scales (4 AA)		
Scale is calibrated regularly or when variance are observed. (May be more or less according to Facility regulations) Use a calibrated 25 or (50lb for Bariatric) weight to do calibrations.		
Inspect Reservoir Tank		

The above items should be adjusted if issues occur more frequently. Each facility is different so quality of water plays an important role in many of these checks. Many of the items listed above are in your manuals. The manuals are a great source of information regarding making adjustments to temperatures installing and operating. Also your facility or state and local codes may require more frequent inspections.