


5. Cleaning and maintenance

The Reflex must be cleaned after every use, with water and regular household cleaning agents. Remove excess soap residues. PUR integrated foam components with IMC coating should be rubbed clean and disinfected. **Do not use any abrasive or other aggressive cleaning products (e.g. chlorine bleach).** Practical experience has shown that regular household cleaning agents with limited alcohol content (max. 5%) are most suitable for this purpose. The product Neoquat combi® is recommended for disinfection purposes (on the basis of long-term testing). If necessary, the PUR components can be cleaned with a solution of 70% ethanol or propanol.

You must ensure that both the disinfecting and cleaning processes involve the components being rubbed and not just submerged. The time the products must be left on the components has a significant impact upon the lifespan of the PUR components. As a result of the material properties of PUR, contact with sharp objects must be avoided. This could lead to damage or tears in the material. Finally, we would like to point out that non-critical parts of the PUR components must also be cleaned and decontaminated.

Check the wheels regularly for hairs, dust and soap residues and remove when necessary. Maintenance and repairs to the Reflex may only be carried out by authorised personnel.

Maintenance schedule	
After every use	<ul style="list-style-type: none"> • Clean chair with water and regular household cleaning products.
Daily	<ul style="list-style-type: none"> • PUR foam components must be cleaned and disinfected. • Check to ensure all parts of the Reflex are present, including the footrests, head support and filler piece for the seat.
Weekly	<ul style="list-style-type: none"> • Clean frame. • Check wheels for hairs, dust and soap residues and remove if necessary. • Check that the chair is still working correctly. • Charge the battery at least once a week.
Annually 	<ul style="list-style-type: none"> • According to the guidelines for medical devices, a chair must be maintained in line with the manufacturer's instructions. This process will preferably be carried out by Lopital or another authorised body. Lopital offers the option of having the chair thoroughly inspected and serviced once a year via Periodic Preventative Maintenance. <p>For more information about the maintenance contract, please contact Lopital (service@lopital.nl) or speak to your supplier.</p>

6. Storage and transport

During transport and storage, the ambient conditions must remain moderate. In other words, the temperature should remain between -40 and +55 degrees Celsius (-40 to 130 degrees Fahrenheit) and humidity levels should be kept as low as possible. Vibration during transport must be kept to a minimum in order to avoid damaging the hinged mechanisms on the chair. The chair must be switched off during transport (*See paragraph 4.1. Lock button*).