

Subject: Vancare authorized Sanitization procedure Using Vital Oxide

The following procedure is the only recommend application for sanitization of the Vancare Sling.

- 1) Launder and dry slings as per instructions on sling and or manual using only mild detergent and appropriate drying temperature.
- 2) Post drying: spray the entire sling with Vital Oxide using the Vancare ESS machine (if not available use hand held Vital Oxide spray bottle) being sure to saturate the all areas of the sling. Let sling sit for 10 minutes before moving or placing back into use.
- 3) Use of any other Chemicals is not authorized By Vancare for sanitization and may cause permanent damage to the sling rendering it unsafe for use.

Please contact us with any questions or concerns.