

Disinfecting Procedure



THE TUB MUST BE UNOCCUPIED AND SAFETY PROCEDURES FOLLOWED.

- Unlock and open the “Disinfectant Cabinet” door.
- Remove the “Disinfectant Wand” from its holder and point spray head into the tub.
- Turn the “Disinfectant Valve” to the “**DISINFECT**” position.
- Spray the interior surface of the tub. Use brush or sponge if needed.
- Turn the “Disinfectant Valve” to the “**OFF**” position.
- Allow disinfectant solution to remain on the interior surface of the tub for the period of time recommended by the manufacturer.
- Turn the “Disinfectant Valve” to the “**RINSE**” position. Thoroughly rinse the interior surface of the tub.
- Turn the “Disinfectant Valve” to the “**OFF**” position. Return the “Disinfectant Wand” to its holder and lock the “Disinfectant Cabinet” door.



Disinfectant
Valve

Disinfectant Cabinet