


Safety recommendations

- Read the instructions for use carefully (find this at: www.procare-medical.com) before use.
- Maximum load 397 LB.
- Make sure that the brake is set during client transfers from and to the device.
- Place both feet on the foot rests.
- Armrests are locked during up / down movements and arms should rest on armrest.
- Only relocate and move the chair in the lowest position practical.

Operation panel

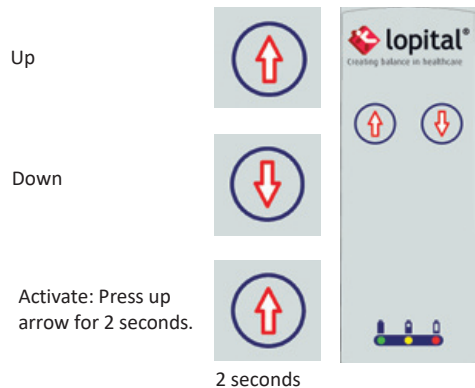

Important!
This instruction card may not be considered a replacement for the instructions for use.




Use

1. Activate the Elexo by pressing the 'up' button for 2 seconds.
Note: After use, the Elexo will go into a sleep setting after 60 minutes.
2. Place the chair in position for transfer and fold the foot supports up in preparation for the transfer.
3. Apply the brakes to the chair!
4. Open the armrests and fold these up.
5. Support the patient during the transfer.
6. Lower the armrests and insure the client's feet rest on the foot plates.

Use the same procedure to transfer the patient from the Elexo. (With the exception of # 6.)

Wheel Brakes

1. Apply Wheel Brake by pressing down on blue tab.
2. Release brake by lifting up blue tab.

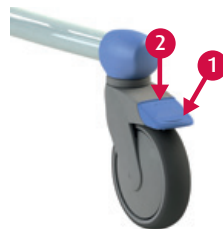


Foot rest



Both foot plates must be turned away for the patient to take a seat in the chair. Make sure that the person places both feet on the foot plates when the chair is moving.

Armrest with front closure

1. Unlocking. (Press up on locking lever).
2. Rotating.
3. Folding up.

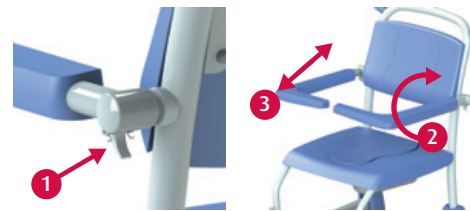

Charging batteries

After using the Elexo, connect it to the charger to maintain the batteries.



- | | |
|-----------------|---------------------------|
| Off | Sleep setting. |
| Green | Chair ready to use. |
| Blinking orange | Emergency stop activated. |
| Orange | Battery is half empty. |
| Blinking red | Battery almost empty. |
| Red | Battery empty. |


The Elexo will not work during charging.

Toilet function

- Remove the inlay on the seat before the person sits down in the chair.
- *In combination with toilet:* The Elexo can be placed over almost all toilets. Move the chair up or down to achieve the right position.
- *With bedpan:* The Elexo can also be used as a toilet chair. Simply slide the stainless steel / plastic bedpan under the chair using the guides.

Emergency lowering

If the Elexo Hand Control malfunctions, you can lower the chair using the emergency lowering button.

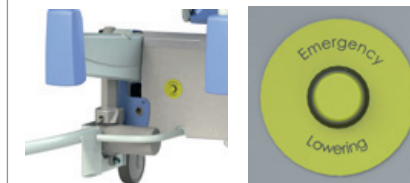

Cleaning and maintenance

The Elexo must be cleaned after every use, with water and regular household cleaning agents. Do not use any abrasive or other aggressive cleaning products. Check the wheels regularly for hairs, dust and soap residues and remove when necessary.

The complete maintenance schedule can be found in the instructions for use ([find this at: www.procare-medical.com](http://www.procare-medical.com)).

If you have any questions or need service contact:

ProCare Medical
855-528-0421

Instructions for use
For the full instructions for use go to www.ProCare-Medical.com