

Safety recommendations

- Read the instructions for use carefully (find this at: www.procare-medical.com) before use.
- Maximum load 297 pounds.
- Make sure that the chair brakes are on during patient transfers from and to the device.
- Place the foot rests at the right height and place the feet on the rests.
- Set head support to correct position (usually by tilting).
- Front armrests are down and locked during up / down movements / transport
- Arms should rest on armrest. Only relocate and move the chair in the lowest position practical.

Operation panel

Up



Down



Tilt forwards



Tilt backwards



Activate




Lock button



Important!

This instruction card may not be considered a replacement for the instructions for use.

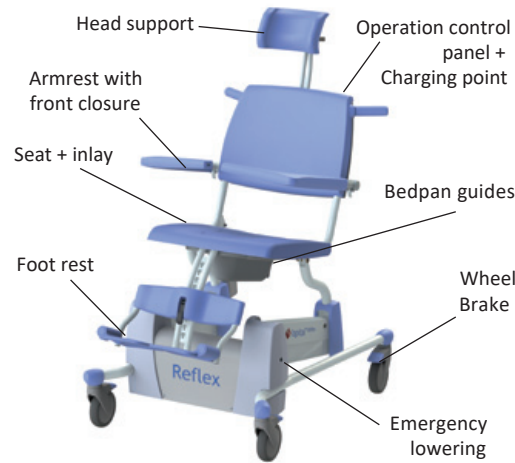

Use

1. Activate the Reflex by pressing the two 'up' buttons simultaneously for 1 second.
Note: After use, the Reflex will go into a sleep setting after 60 minutes.
2. Move the chair to the best position for the transfer and adjust the head and foot rests.
3. Apply the brakes to the chair!
4. Open the armrests and fold these up if necessary.
5. Support the patient during the transfer.


6. Close the armrests.

Follow the same steps to transfer the patient from the Reflex (except step #6).


All functions on the Reflex are halted as a result of using the lock button. The Reflex must be reset. You do this by pressing both 'up' buttons for 4 seconds.

Wheel Brakes

1. Apply Wheel Brake by pressing down on blue tab.
2. Release brake by lifting up blue tab.

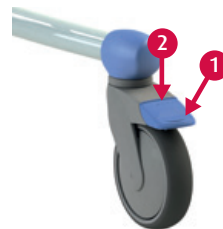


Foot rest

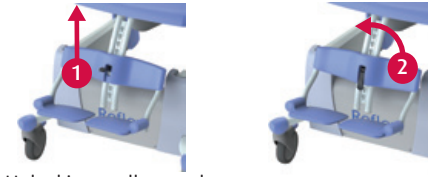

1. Unlocking- pull up on lever.
2. Remove complete foot rest.
3. Choose desired position.
4. Hook in and lock by pushing down on lever.

Head support

1. Set height of head rest.
Loosen locking wheel, tighten wheel when correct height is set.
2. Set depth of headrest.
3. Loosen locking lever, tighten locking lever when correct depth is set.

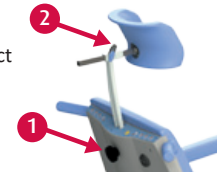

Armrest with front closure

1. Unlocking. (Press up on locking lever).
2. Rotating.
3. Folding up.



Charging batteries

After using the Reflex, connect it to the charger to maintain the batteries.



- | | |
|--------------|-----------------------|
| Off | Sleep setting. |
| Green | Chair ready to use. |
| Blinking red | Battery almost empty. |
| Red | Battery empty. |
- The Reflex will not work during charging.

Toilet function

- Remove the inlay on the seat before the person sits down in the chair.
- *In combination with toilet:* The Reflex can be moved over almost every toilet. Move the chair upwards or downwards to achieve the right position.
- *With bedpan:* The Reflex can also be used as a toilet chair. Simply slide the stainless steel / plastic bedpan under the chair using the guides.

Emergency lowering

If the Reflex fails to move with the controls, you can lower the chair using the emergency lowering handle.



Cleaning and maintenance

The Reflex must be cleaned after every use, with water and regular household cleaning agents. Do not use any abrasive or other aggressive cleaning products. Check the wheels regularly for hairs, dust and soap residues and remove when necessary.

The complete maintenance schedule can be found in the instructions for use ([find this at: www.procare-medical.com](http://www.procare-medical.com)).

If you have any questions or need service contact:

ProCare Medical
855-528-0421

Instructions for use
For the full instructions for use go to:
www.ProCare-Medical.com