




Use

1. Place the chair in position for transfer and rotate the foot rests away in preparation for the transfer.
2. Apply the brakes to the chair!
3. Raise the armrests if necessary.
4. Support the patient during the transfer.
5. Lower the armrests and insure the patient's feet rest on the foot rests.

Use the same procedure to transfer the patient from the Tango. (With the exception of # 5.)


Wheel blocking

1. Apply Wheel Brake by pressing down on blue tab.
2. Release brake by lifting up blue tab.

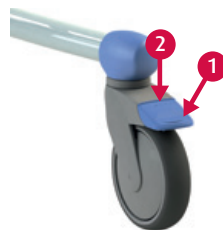

Directional / braked wheel

1. To block the wheel direction (1x).
Wheel blocking (2x).
2. Unlocking.

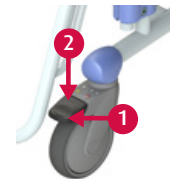

Foot rest



- 1 Both foot rests must be turned away (1) for the patient to take a seat in the chair. Turn footrests back into position when person is seated (2). Make sure that the person places both feet on the foot rests when the chair is moving.

Armrest

1. Flip up armrests.



Toilet function

- Remove the lid on the seat before the person sits down in the chair.
- *In combination with toilet:* The Tango XL can be moved over almost every toilet.
- *With bedpan:* The Tango XL can also be used as a toilet chair. Simply slide the stainless steel / plastic bedpan under the chair using the guides.

Cleaning and maintenance

The Tango XL must be cleaned after every use, with water and regular household cleaning agents. Do not use any abrasive or other aggressive cleaning products. Check the wheels regularly for hairs, dust and soap residues and remove when necessary.

The complete maintenance schedule can be found in the instructions for use (*find this at: www.lopital.com*).

If you have any questions or need service contact:

ProCare Medical
855-528-0421

Instructions for use

For the full instructions for use go to www.lopital.com

Safety recommendations

- Read the instructions for use carefully (find this at: www.procare-medical.com) before use.
- Maximum load 550 Lbs.
- Make sure that the the brake is set during patient transfers from and to the device.
- Place both feet on foot rests and arms on arm rests prior to moving Tango with resident.

Important!

This instruction card may not be considered a replacement for the instructions for use.